



TESTING KITS FOR DRUGS

## Procedure Card for IDT0950

# METHYLPHENIDATE (RITALIN) IDENTIFICATION

## Substance Test



### Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

|                  |   |   |   |   |
|------------------|---|---|---|---|
| Test Preparation | 1 | Remove the Smart-Tip™ probe from test unit. | 2 Sample suspected substance.   |   |
|                  |   |   | <b>Powder</b><br>Touch Smart-Tip™ probe to sample. Make sure the tip is completely covered.<br> | <b>Tablet</b><br>Break the tablet and insert a piece, <b>about the size of the hole (4 mm)</b> , into the opening of the kit.<br> |

|                  |   |   |  |                |   |  |   |   |
|------------------|---|---|--|----------------|---|--|---|---|
| Test Preparation | 3 | Lock Smart-Tip™ probe into test unit. Press firmly. | <b>STOP</b><br><b>Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.</b> | Identification | 4 | With <b>thumb and forefinger</b> , squeeze chamber 1 until capsule breaks. | 5 | Gently shake test unit for 2-3 seconds. |
|                  |   |   |  |                |   |  |   |   |

|         |  |   |
|---------|--|---|
| Results |  | The appearance of turquoise color particles adhering to the sampler tip or precipitating out of solution indicates the presence of <b>Methylphenidate (Ritalin)</b> . |
|         |  | * Note: Color may disappear with time. The liquid will remain pink.<br><br>If no turquoise color develops, Methylphenidate (Ritalin) is <b>not</b> present.           |



### Disposal

The test kit is safe to dispose of in the trash.

01-19