



TESTING KITS FOR DRUGS

Procedure Card for IDT2200

MIX HEROIN IDENTIFICATION

Substance Test



As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1 Remove the Smart-Tip™ probe from test unit.



Testing Options

If the suspected substance is a brown powder, proceed to Steps 2-5 on Side 1.

If the suspected substance is a white powder or a tablet, proceed to Steps 6-11 on Side 2.

Brown Heroin Test

Brown Heroin Preparation


2 Sample suspected substance.

Brown Powder

Touch Smart-Tip™ probe to sample.




3 Lock Smart-Tip™ probe into test unit. Press firmly.

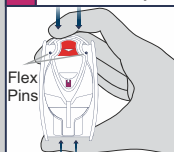


STOP
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Brown Heroin Identification


4 With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

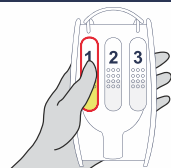


5 Gently shake test unit for 2-3 seconds.

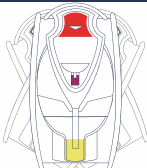

Brown Heroin Results

 If the substance is a brown powder, the appearance of a purple-red color strongly indicates the presence of **Brown Heroin**.

 If no purple-red color develops, Brown Heroin is not present. Regardless of whether or not any color developed, do not continue to White Heroin testing. **STOP** and proceed to Disposal.



Disposal The test kit is safe to dispose of in the trash.

01-19



Manufactured by IDenta Ltd
www.identa-corp.com
Patented Technology • Made in Israel



TESTING KITS FOR DRUGS

Procedure Card for IDT2200

MIX HEROIN IDENTIFICATION

Substance Test

SIDE 2

White Heroin Test

White Heroin Preparation

6 Sample suspected substance.

Powder

Touch Smart-Tip™ probe to sample.




Tablet

Firmly press Smart-Tip™ probe on the tablet while scrubbing in a circular motion until the tip is completely covered.



7 Lock Smart-Tip™ probe into test unit. Press firmly.

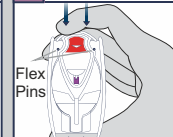


STOP
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.


White Heroin Identification

8 With **thumb and forefinger**, squeeze chamber 2 until capsule breaks.

DO NOT BREAK CHAMBER 1

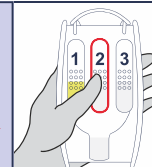

9 Gently shake test unit for 2-3 seconds.

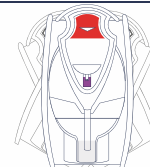

White Heroin Results



If the substance is a powder, the appearance of a purple color indicates the presence of an opiate such as **Heroin**.
If the substance is a tablet, the appearance of a purple color indicates the presence of an opiate such as **Morphine** or **Codeine**.



If no purple color develops, an opiate, such as Heroin or Morphine, is not present. Regardless of whether or not any color developed, do not continue to Brown Heroin testing. **STOP** and proceed to Dilution & Disposal.




Proceed to Dilution (steps 10-11), regardless of whether or not a color appeared.




Dilution

10 With **thumb and forefinger**, squeeze chamber 3 until capsule breaks.

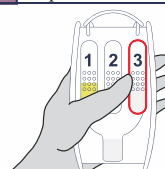

11 Gently shake test unit for 2-3 seconds.

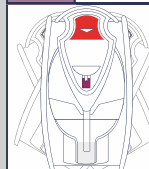

The test window will warm up. This is normal.



Disposal After Dilution, the test kit is safe to dispose of in the trash.

01-19



Manufactured by IDenta Ltd
www.identa-corp.com
Patented Technology • Made in Israel