



TESTING KITS FOR DRUGS

Procedure Card for IDT9000


GENERAL SCREENING DRUGS

Substance Test

Read Before Use


As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

1 Remove the Smart-Tip™ probe from test unit.




2 Sample suspected substance or surface. (Not for use with plant materials.)

Powder	Tablet	Encapsulated Powder
Touch Smart-Tip™ probe to sample.	Firmly press Smart-Tip™ probe on the tablet while scrubbing in a circular motion until the tip is completely covered.	Safely open shell. Gently pour substance out and touch Smart-Tip™ to sample.

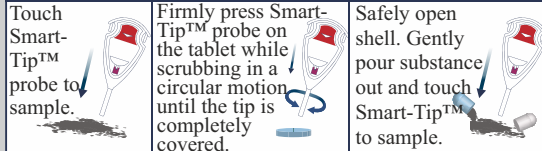


3 Lock Smart-Tip™ probe into test unit. Press firmly.

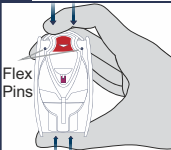


STOP
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.


4 With thumb and forefinger, squeeze chamber 1 until capsule breaks.

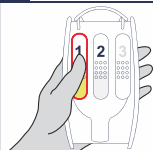



5 Gently shake test unit for 2-3 seconds.




Chamber 1 Results

 If a brown powder is present, the appearance of a purple-red color strongly indicates the presence of **Brown Heroin**.
If a brown powder is not present, the appearance of a purple-red color indicates the presence of a suspected drug (See List below).


 The appearance of a green or blue-green color indicates the presence of **Crack (Cocaine Base)**.

 If any of the above colors develop, proceed to Disposal. If none of the above colors develop, proceed to Step 6.


6 With thumb and forefinger, squeeze chamber 2 until capsule breaks.

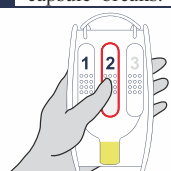

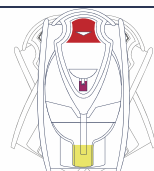

7 Gently shake test unit for 2-3 seconds.



Chamber 2 Results

 The appearance of a purple-red color indicates the presence of a **basic alkaloid substance (possible drug - See List below)**.

 The appearance of a green or blue-green color indicates the presence of **Cocaine**.

 If none of the above colors develop, the above drugs and the drugs in the List are **not** present.

List

If the suspected substance is a powder, crystals, granules, or flakes, or an illicit tablet (an unusual tablet marked with a symbol), or an illicit pill, then one of these drugs may be present:

Heroin, Amphetamine, Methamphetamine, Ecstasy/MDMA, Buphedrone, Cathinone, DMT, Fentanyl, Ketamine, Mephedrone, Methadone, MethCathinone, Methylone, mCPP, MDPBP, MDPV, MPA, PCP, PMA, PMMA, 2-aminoindane, other synthetic/designer drugs.

Caution: Basic alkaloids are also present in some legal medications.

Disposal The test kit is safe to dispose of in the trash.

