



TESTING KITS FOR DRUGS

Procedure Card for IDT0750

PARACETAMOL IDENTIFICATION

Substance Test




Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation


1

Remove the Smart-Tip™ probe from test unit.

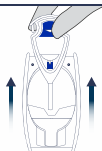


2

Sample suspected substance.

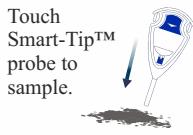

3

Lock Smart-Tip™ probe into test unit. Press firmly.

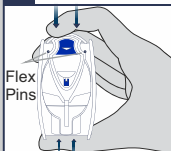


STOP

Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Identification


4

With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

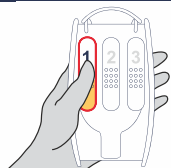

5

Gently shake test unit for 2-3 seconds.

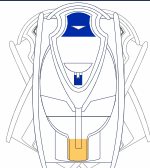

Results



The appearance of a blue color indicates the presence of **Paracetamol**.



If no blue color develops, Paracetamol is not present.



Disposal

The test kit is safe to dispose of in the trash.

01-19