



As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.



For Substance Only: This test is designed to enhance positive results from Cocaine/Crack Identification Kit (IDT1000T). Only proceed to use this test after obtaining a turquoise result from the Identification Kit.

Test Preparation	1	Remove the Smart-Tip™ probe from test unit.	2	Sample suspected substance.	3	Lock Smart-Tip™ probe into test unit. Press firmly.	<p>STOP</p> <p>Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.</p>

Testing Options	If confirming Crack, proceed to steps 4-5.	
	If confirming Cocaine, proceed to steps 6-8.	

Crack Confirmation	4	With thumb and forefinger, squeeze chamber 1 until capsule breaks.	5	Gently shake test unit for 2-3 seconds.

Crack Confirmation Results		The appearance of a green or blue-green color strongly indicates the presence of Crack (Cocaine Base) , if Crack was indicated with Cocaine/Crack Identification Kit (IDT1000T).
		If no green or blue-green color develops, Crack is not present. Regardless of whether or not a color develops, proceed to Disposal.

Cocaine Confirmation	6	With thumb and forefinger, squeeze chamber 1 until capsule breaks.	7	With thumb and forefinger, squeeze chamber 2 until capsule breaks.	8	Gently shake test unit for 2-3 seconds.

Cocaine Confirmation Results		The appearance of a green or blue-green color strongly indicates the presence of Cocaine , if Cocaine was indicated with Cocaine/Crack Identification Kit (IDT1000T).
		The appearance of a purple-red color indicates the presumed presence of a basic alkaloid substance (suspected drug).
		If none of the above colors develop, neither Cocaine nor basic alkaloids are present.

	Disposal	The test kit is safe to dispose of in the trash.
--	-----------------	--